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Vaginal dryness is one of the side effects that can affect the quality of life of women undergoing chemotherapy and radiotherapy. **Intimate Comfort Pessaries** (medical device) were created to help the concerned women take care of their intimate well-being and femininity.

Intimate Comfort Pessaries (medical device) restore hydration in case of vaginal dryness or atrophy due to chemotherapy and radiotherapy treatments, menopause, diabetes, postpartum period or stress for effective and lasting relief. Soothes irritations, itching, burning and dyspareunia, caused by poor or absent vaginal secretion.

Directions for use: wash your hands carefully, extract one single ovule and insert it deeply into the vagina, in a supine position, before going to bed.

Dosage: Intensive treatment: 1 ovule a day for 7 days. Maintenance treatment: 1 ovule every 3 days or according to your doctor's advice.

Warnings: avoid contact with eyes; in case of contact, wash with plenty of water. Keep out of reach of children. Do not swallow. Store in a cool and dry place, away from heat sources (at temperature lower than 25°C). Do not use after the expiry date indicated on the box. Expiry date refers to the product properly stored. Do not use if the package is opened or damaged. Do not use on damaged mucous membrane. Do not use if you have a known hypersensitivity or allergy to any of the ingredients. For use during pregnancy and breast feeding period please consult your doctor.

INGREDIENTS: Hyaluronic acid (sodium salt), Vitamin E, Vitamin A, lipophilic extract of Aloe vera, semisynthetic glycerides.

EKUBERG PHARMA
Via Tito Schipa 6, Carpignano Salentino (LE) - Italy
Certified company ISO 9001

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